


Philips NeoLead Electrodes

Application and Removal Instructions:

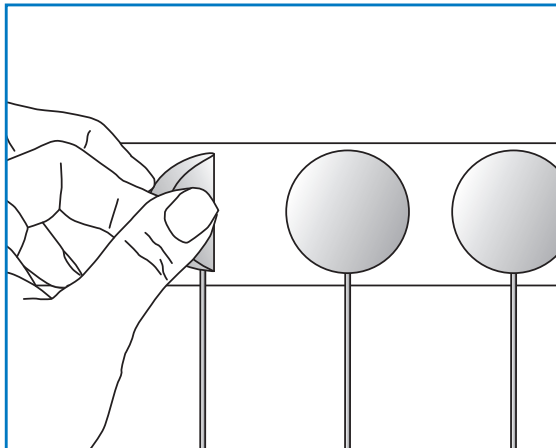


- 1** Skin must be intact, clean and dry. No skin prep is recommended. Moisture, oils or lotions adversely affect the adhesion of hydrocolloid.
DO NOT USE ALCOHOL.

NOTE: While clear liner is in place, warm product in hands for 60 seconds to activate adhesive. Product may also be held near radiant warmer or heel warmer.

- 2** Remove NeoLead from liner.



- 3** Place NeoLead on skin where it is to be secured.
- 4** Hold in place for 60 seconds to ensure proper adhesion. Heat will increase the bond between the product and patient.
- 5** Attach leadwire to ECG Trunk Cable.

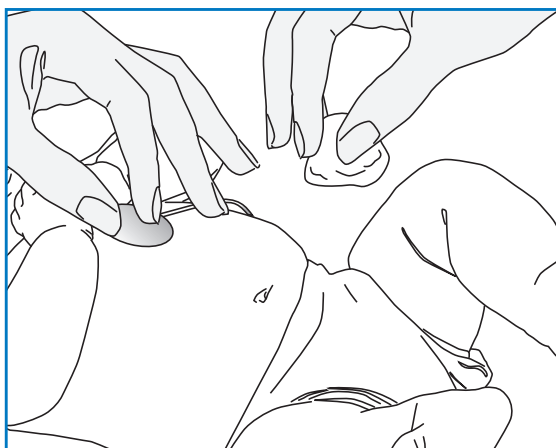

- 6** To remove, saturate surface of electrode with water and gently peel away from skin as you continue to swab with water. Soap may be used for removal.
- 7** Change every 5–7 days or per hospital protocol.

NOTE: Discontinue immediately if skin irritation occurs.

Please see product instructions for use for a complete listing of contraindications, warnings, and precautions.